**APPROVED SOLUTIONS LIST**

English  
Page 1

English  
Page 2

English  
Page 3

English  
Page 4

French  
Page 5

French  
Page 6

German  
Page 7

Italian  
Page 8

Click language and region to jump to relevant page.

English

Contamac has produced this list of contact lens care solutions that are compatible with the Tangible Hydra-PEG™ coating on lenses made in the Optimum GP materials.

**USE ONLY APPROVED TANGIBLE HYDRA-PEG™ CLEANERS**

Multipurpose Solutions

|  |                  |
|--|------------------|
| Regard® K RGP Contact Lens Solution              | by Vita Research |
| Boots Total                                      | by Boots         |
| Asda All-in-One                                  | by Asda          |
| Boston Simplus®                                  | by Bausch & Lomb |
| Solocare Aqua                                    | by Ciba Vision   |
| Perfect® cleaner/ storage solution for GP lenses | by MPG&E         |
| Menicare Plus                                    | by Menicon       |
| Menicare Pure                                    | by Menicon       |
| Oté Fine   | by Oté Pharma    |
| Oté Clean  | by Oté Pharma    |

|                                  |                |
|----------------------------------|----------------|
| GPHCL                            | by Contopharma |
| Conventional Cleaner /Rinsing    | by Contopharma |
| Total Care Cleaner               | by Abbott      |
| Total Care Disinfection /Storage | by Abbott      |
| GP Multi                         | by Avizor      |

Hydrogen Peroxide Solutions

|                                   |                 |
|-----------------------------------|-----------------|
| Clear Care®                       | by Alcon        |
| Clear Care® Plus with HydraGlyde® | by Alcon        |
| AO Sept® Plus                     | by Alcon        |
| AO Sept® Plus with HydraGlyde®    | by Alcon        |
| Tab-in-One                        | by Contopharma  |
| Refine One Step                   | by Coopervision |
| Monosept 120+                     | by Oté Pharma   |
| Ever Clean                        | by Avizor       |

Enzymatic Cleaners

|   |            |
|---|------------|
| Amclair Weekly Enzymatic Cleaning Tablets | by Abatron |
|---|------------|

**DO NOT USE TAP WATER, ABRASIVE OR ALCOHOL BASED CLEANERS**

(These will damage or destroy the Tangible Hydra-PEG™ coating)

Abrasive Cleaners

|                 |                  |
|-----------------|------------------|
| Boston® Advance | by Bausch & Lomb |
|-----------------|------------------|

or similar products

Alcohol Based Cleaners

|                                  |                      |
|----------------------------------|----------------------|
| Miraflo (and equivalents)        | by various companies |
| i-clean!                         | by Contopharma       |
| Boots Advanced Cleaning Solution | by Boots             |

or similar products

Others

|  |                  |
|--|------------------|
| Progent                                  | by Menicon       |
| Oté Twins Active                         | by Oté           |
| Boston One Step Liquid Enzymatic Cleaner | by Bausch & Lomb |
| Abatron Quattro                          | by Abatron       |

or similar products

We advocate following the instructions indicated with each of the care solutions, including, where stated, rubbing and rinsing the lenses where appropriate.

Lenses should not be stored dry, but always in a contact lens care solution, according to the instructions on the packaging of that solution.

You should never rub the lenses with a dry tissue or cloth. These are often very abrasive at a surface level and can damage the Tangible Hydra-PEG™ coating.

**APPROVED SOLUTIONS LIST**

English  
Page 1

English  
Page 2

English  
Page 3

English  
Page 4

French  
Page 5

French  
Page 6

German  
Page 7

Italian  
Page 8

Click language and region to jump to relevant page.

English

Contamac has produced this list of contact lens care solutions that are compatible with the Tangible Hydra-PEG™ coating on lenses made in the Optimum GP materials.

**USE ONLY APPROVED TANGIBLE HYDRA-PEG™ CLEANERS**

**Multipurpose Solutions**

Menicon Unique pH by Menicon  
Boston Simplus® by Bausch & Lomb

**Hydrogen Peroxide Solutions**

Clear Care® by Alcon  
Clear Care® Plus with HydraGlyde® by Alcon

**DO NOT USE TAP WATER, ABRASIVE OR ALCOHOL-BASED CLEANERS**

(These will damage or destroy the Tangible Hydra-PEG™ coating)

**Abrasive Cleaners**

Boston® Advance by Bausch & Lomb

or similar products

**Alcohol Based Cleaners**

Miraflow (and equivalents) by various companies

or similar products

**Others**

Progent by Menicon  
Boston One Step Liquid Enzymatic Cleaner by Bausch & Lomb  
Optimum by Lobob ESC Extra Strength Cleaner by Lobob Laboratories

or similar products

We advocate following the instructions indicated with each of the care solutions, including, where stated, rubbing and rinsing the lenses where appropriate.

You should never rub the lenses with a dry tissue or cloth. These are often very abrasive at a surface level and can damage the Tangible Hydra-PEG™ coating.

Lenses should not be stored dry, but always in a contact lens care solution, according to the instructions on the packaging of that solution.

**APPROVED SOLUTIONS LIST**

English  
Page 1

English  
Page 2

English  
Page 3

English  
Page 4

French  
Page 5

French  
Page 6

German  
Page 7

Italian  
Page 8

Click language and region to jump to relevant page.

English

Contamac has produced this list of contact lens care solutions that are compatible with the Tangible Hydra-PEG™ coating on lenses made in the Optimum GP materials.

**USE ONLY APPROVED TANGIBLE HYDRA-PEG™ CLEANERS**

**Multipurpose Solutions**

|                                     |                  |
|-------------------------------------|------------------|
| Regard® K RGP Contact Lens Solution | by Vita Research |
| Boston Simplus®                     | by Bausch & Lomb |
| Solocare Aqua                       | by Ciba Vision   |
| Menicare Plus                       | by Menicon       |
| Total Care Cleaner                  | by Abbott        |
| Total Care Disinfection /Storage    | by Abbott        |

**Hydrogen Peroxide Solutions**

|                                   |          |
|-----------------------------------|----------|
| Clear Care®                       | by Alcon |
| Clear Care® Plus with HydraGlyde® | by Alcon |
| AO Sept® Plus                     | by Alcon |
| AO Sept® Plus with HydraGlyde®    | by Alcon |

**DO NOT USE TAP WATER, ABRASIVE OR ALCOHOL BASED CLEANERS**

(These will damage or destroy the Tangible Hydra-PEG™ coating)

**Abrasive Cleaners**

|                 |                  |
|-----------------|------------------|
| Boston® Advance | by Bausch & Lomb |
|-----------------|------------------|

or similar products

**Alcohol Based Cleaners**

|                           |                      |
|---------------------------|----------------------|
| Miraflo (and equivalents) | by various companies |
|---------------------------|----------------------|

or similar products

**Others**

|  |                  |
|--|------------------|
| Progent                                  | by Menicon       |
| Boston One Step Liquid Enzymatic Cleaner | by Bausch & Lomb |

or similar products

We advocate following the instructions indicated with each of the care solutions, including, where stated, rubbing and rinsing the lenses where appropriate.

You should never rub the lenses with a dry tissue or cloth. These are often very abrasive at a surface level and can damage the Tangible Hydra-PEG™ coating.

Lenses should not be stored dry, but always in a contact lens care solution, according to the instructions on the packaging of that solution.

**APPROVED SOLUTIONS LIST**

English  
Page 1

English  
Page 2

English  
Page 3

English  
Page 4

French  
Page 5

French  
Page 6

German  
Page 7

Italian  
Page 8

Click language and region to jump to relevant page.

English

Contamac has produced this list of contact lens care solutions that are compatible with the Tangible Hydra-PEG™ coating on lenses made in the Optimum GP materials.

**USE ONLY APPROVED TANGIBLE HYDRA-PEG™ CLEANERS**

**Multipurpose Solutions**

Menicon Unique pH by Menicon  
Boston Simplus® by Bausch & Lomb  
Solocare Aqua by Ciba Vision

**Hydrogen Peroxide Solutions**

Clear Care® by Alcon  
Clear Care® Plus with HydraGlyde® by Alcon

**Enzymatic Cleaners**

Amclair Weekly Enzymatic Cleaning Tablets by Abatron

**DO NOT USE TAP WATER, ABRASIVE OR ALCOHOL BASED CLEANERS**

(These will damage or destroy the Tangible Hydra-PEG™ coating)

**Abrasive Cleaners**

Boston® Advance by Bausch & Lomb

or similar products

**Alcohol Based Cleaners**

Miraflow (and equivalents) by various companies

or similar products

**Others**

Progent by Menicon  
Boston One Step Liquid Enzymatic Cleaner by Bausch & Lomb

or similar products

We advocate following the instructions indicated with each of the care solutions, including, where stated, rubbing and rinsing the lenses where appropriate.

You should never rub the lenses with a dry tissue or cloth. These are often very abrasive at a surface level and can damage the Tangible Hydra-PEG™ coating.

Lenses should not be stored dry, but always in a contact lens care solution, according to the instructions on the packaging of that solution.

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English  
Page 1

English  
Page 2

English  
Page 3

English  
Page 4

French  
Page 5

French  
Page 6

German  
Page 7

Italian  
Page 8

Click language and region to jump to relevant page.

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